


Quick reference Guide Type 4
INFRA RED HEARING ASSISTANCE

William Sound WIR RX22-4 Receiver

OPERATING INSTRUCTIONS

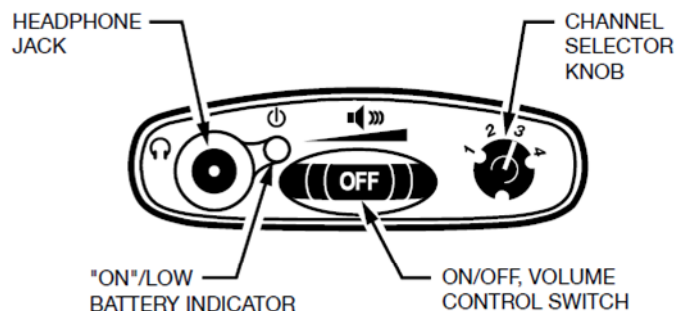

1. Insert earphone or neckloop into the headphone jack.
Neckloops available for loan work with hearing aids equipped with T-Switch (telephone coil). User supplied 3.5mm headphones can also be used.
2. Turn the receiver on by rotating the Volume control knob.
The ON indicator should illuminate RED.
3. Select channel 1 by turning the channel selector knob on the control panel.


IMPORTANT: Make sure the “SoundPlus[®] IR” artwork on the front of the unit is not covered while in use. Do not place the receiver in a pocket or purse, or it will not work properly.

4. Adjust the volume to a comfortable listening level.
5. To turn the receiver off, rotate the Volume control knob to the left until it clicks off. The ON indicator should not be lit. To prolong the battery life of the unit, turn the receiver off when it is not in use.

Quick reference Guide Type 4
INFRA RED HEARING ASSISTANCE

William Sound WIR RX22-4 Receiver

OPERATING INSTRUCTIONS

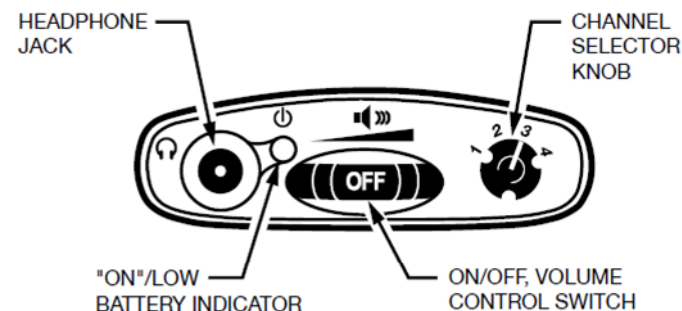

1. Insert earphone or neckloop into the headphone jack.
Neckloops available for loan work with hearing aids equipped with T-Switch (telephone coil). User supplied 3.5mm headphones can also be used.
2. Turn the receiver on by rotating the Volume control knob.
The ON indicator should illuminate RED.
3. Select channel 1 by turning the channel selector knob on the control panel.

IMPORTANT: Make sure the “SoundPlus[®] IR” artwork on the front of the unit is not covered while in use. Do not place the receiver in a pocket or purse, or it will not work properly.

4. Adjust the volume to a comfortable listening level.
5. To turn the receiver off, rotate the Volume control knob to the left until it clicks off. The ON indicator should not be lit. To prolong the battery life of the unit, turn the receiver off when it is not in use.